

earclin ITCH

PLEASE READ THIS INFORMATION CAREFULLY before using earClin itch.
DO NOT THROW THIS INSTRUCTION FOR USE AWAY, it contains important information.

1. **USE FOR**
2. **HOW TO USE**
3. **CARRIERS OF HEARING AIDS**
4. **PRECAUTIONS**
5. **INGREDIENTS**
6. **PACKAGING**
7. **TIPS**
8. **CONTACT DETAILS**

1. USE FOR

Itching in the ear canal caused by dry skin, helps to prevent swimmer's ear and excessive wax. Also suitable for carriers of hearing aids and Earplugs.

- **ITCHING** The oil soothes dry skin in the ear canal and thus relieves itching.
- **HELPS TO PREVENT SWIMMER'S EAR:** The spray contains a natural oil that forms a protective layer in the ear canal.
- **EXCESSIVE EARWAX:** the oil helps soften the earwax. This supports the natural drainage of earwax and thus prevents excessive wax. On later age ear wax can thicken faster, causing older people possible suffering from dry, friable earwax.

NATURAL OILS

earClin Itch contains a combination of natural coconut oil and sunflower oil.

2. HOW TO USE EASY DOSAGE SPRAY

This spray is easy to use: because of the easy dosing spray, the oil gets directly to the right place.

DOSAGE AND USE

Adults: 1-2x per day, one spray per ear.
Children from 12 years: 1x per day, one spray per ear.

If necessary, the ear spray can be used for a longer period of time. Without interruption use up to 30 days.

Use of a cold liquid may feel unpleasant. Therefore, use the ear spray at room temperature.

HOW TO USE

1. Remove the plastic, transparent cap from the bottle.
2. Place the enclosed spray cup on the bottle. When using the spray for the first time, pump the air out of the device (3-4 times pumps).
3. Shake the bottle.
4. Hold the bottle upright and place the spray head against the ear canal. Push the spray head (1 second) so that the oil is sprayed in the ear. At the last remnant of oil, keep the bottle at an angle of 45 degrees, so the end of the tube can't suck up air. A small amount of oil can remain in the bottle.
5. Massage the bottom of the ear (10 seconds).
6. Keep the head tilted, so that excess oil can drain from the ear and pat with a cotton ball.

If you lie down after using the spray, excess oil may drain from the ear. To prevent stains, put a towel under your head.

Clean the spray head with warm water after use. Because of hygienic reasons, the ear spray should only be used by one and the same person.

3. CARRIERS OF HEARING AIDS

In the evening before bedtime, after removing the hearing aid, spray once in the ear. Use of the ear spray does not adversely affect the hearing aid.

CARRIERS OF EARPLUGS

After removing the earplugs, spray once in the ear.

4. PRECAUTIONS

- Do not exceed the indicated dosage.
- Don't apply other products in the ear canal the first half hour after using the spray.
- The ear always has to be dry before hearing aids or earplugs are applied.
- Consult a GP for children aged 12 and older to ensure that ear problems don't require a different medical treatment.
- Keep out of the reach and sight of children.
- Do not use after the expiration date.
- Contact a GP if symptoms persist for more than 30 days.

USE DURING PREGNANCY OR LACTATION

This product can, as far as known, be used when used in the right dosage.

NOT TO BE USED IF

- Hypersensitivity to one of the components or if you currently suffer from ear infection, ear pain, eardrum perforation or if you wear eardrum tubes.
- Under the age of 12

TOLERABILITY

In the recommended quantities, the product is generally well tolerated.

There are no known side effects.

5. INGREDIENTS

Medium triglycerides from coconut, Helianthus annuus (sunflower oil). After opening, the product can be used for 6 months.

6. PACKAGING

The bottle contains 20 ml of oil, good for about 200 single, measured doses.

7. TIPS

- Do not use cotton swabs or other small objects to clean the ear canals.
- Clean the outside of the ear only with toilet paper or a tissue.
- Chewing movements can support the removal of earwax.

Besides earClin Itch there is also earClin earshower available to prevent excessive earwax from building up and to keep the ear canals clean.

earClin itch is a medical device class I, read the instructions for use before use.

8. CONTACT DETAILS

Distributor

TheOTClab
Paulus Potterstraat 20-III
1071 DA Amsterdam
The Netherlands
www.theotclab.com

Manufacturer

TheOTClab
Paulus Potterstraat 20-III
1071 DA Amsterdam
The Netherlands
www.earclin.com

Last revision date: [May 2018](#).



20ml



EARITCH/TI