

earClin earshower is a medical device. Please read the instructions for use carefully before you start using earClin earshower. Do not throw this leaflet away.

1. WHY DO YOU NEED TO CLEAN YOUR EARS?
2. WHAT IS EARCLIN EARSHOWER?
3. HOW TO USE EARCLIN EARSHOWER?
4. CAN EARCLIN EARSHOWER BE USED BY PEOPLE WHO WEAR A HEARING AID?
5. ARE THERE RISKS ASSOCIATED WITH THE USE OF EARCLIN EARSHOWER?
6. WHY IS EARCLIN EARSHOWER SAFE TO USE?
7. WHY IS THE USE OF COTTON SWABS NOT ADVISED?
8. CONTENT
9. CONTACT DETAILS

## 1. WHY DO YOU NEED TO CLEAN YOUR EARS?

Earwax, or cerumen, is a self-cleaning agent that your body produces. It collects dirt, bacteria, and other debris. Usually, the wax works its way out of the ears naturally through chewing and other jaw motions.

Earwax can build up and affect your hearing. When earwax reaches this level, it's called impaction.

If you have impaction, you may experience symptoms like:

- Aching in the affected ear
- Fullness or ringing in the ear
- Impaired hearing in the affected ear
- An odor coming from the affected ear
- Dizziness
- A cough

You may be more likely to develop excess wax if you use hearing aids or ear plugs. Older adults and people with developmental disabilities have a bigger chance of developing excess earwax. Your ear canals shape may make the natural removal of wax difficult.

## 2. WHAT IS EARCLIN EARSHOWER?

earClin earshower is newly developed based on the same technique used by ear specialists. EarClin earshower contains a unique soap which simply flushes out excess earwax.

It gently rinses your ear canal using water enriched with unique soap. A simple method for cleaning ears. earClin earshower is an effective solution for regularly cleaning the ear canals at home. Like brushing your teeth, it should be part of your personal health and hygiene. Prevent infections and improve your hearing: Use the soapy solution in earClin earshower to simply flush out excess earwax.

earClin earshower is particularly helpful for:

- Athletes with regular exposure to water
- Persons with heavy production of cerumen
- Persons who wear hearing aids, ear protectors, or in-ear earphones
- Persons with regular exposure to dust or dirt

## 3. HOW TO USE EARCLIN EARSHOWER?

earClin earshower is indicated for home use for:

- Persons with hearing aids
- Persons who want to keep their outer ear canal clean
- Adults for self-application
- Children (3 years), applied by an adult

The best time to use earClin earshower is during your usual shower or bath routine. First time use: Flush out earClin earshower 2 times with clear water, after that suck lukewarm water (max 400) into the device, flush the ear, then dry. Like the professional cleaning method used by specialists, earClin earshower cleans the ears with natural, lukewarm water. The soap stick inside the tube not only facilitates the cleaning process but also indicates how much longer the earClin earshower device can be used for. The blue soap will disappear after about 30 days. From that point on, earClin earshower should not be used again and the device should be replaced by a new one.

### How to use earClin earshower?

**Suction bulb:** Use the suction bulb to draw lukewarm water (max 400) up into the earClin earshower and to gently release it into the ear canal. Then use the empty earClin earshower to blow air into the ear canal, removing any remaining moisture.

**Spray nozzle:** Place the spray nozzle at the entrance to the ear canal. It's shape and size stops you from accidentally pushing the earClin earshower too far into your ear.

**Gentle and effective soap:** When mixed with lukewarm water, the ingredients contained in the earClin earshower produce a gentle but effective soapy solution. Once the blue soap in the device disappears, your earClin earshower is empty and needs replacing.

3 applications per ear are recommended. Stubborn deposits inside the ear can only be removed after using earClin earshower for several days.

## 4. CAN EARCLIN EARSHOWER BE USED BY PEOPLE WHO WEAR A HEARING AID?

earClin earshower is particularly intended for people who wear a hearing aid for its regular use (use each day before going to bed) allows the perfect hygiene of the ear canal which is indispensable to the proper operation of the device.

## 5. ARE THERE RISKS ASSOCIATED WITH THE USE OF EARCLIN EARSHOWER?

earClin earshower is well tolerated and is specific developed for all people, above the age of 3 years old.

Stop the use of earClin earshower if there is any allergic reaction. Do not use if you are allergic to one of the ingredients.

**⚠ Do not use without medical opinion in case of otitis, pain in the ear, a perforated ear drum and generally, by any person subject to dizziness or with medical antecedents relating to the ear or ear drum. Without knowing it, you may have a cerumen impaction and it is possible that under the action of earClin earshower this impaction swells causing discomfort. In this case, consult a physician.**

- earClin earshower should not be used on children aged 0-3 or in the case of inflamed ear canals or ear drum conditions.
- earClin earshower must be used by adult persons only. The adult version is intended for self-application for adults, or application in an adult by another adult. The Kids version is intended for application in children, supported by an adult. Keep away from children below 3 years. Small parts present a choking hazard – keep out of reach of children.

### earClin earshower must not be applied

- In persons with pre-damage (e.g., perforation) or deformation of the ear drum
- In persons with pre-damaged ear canal
- In children under 3 years of age

Use earClin earshower max. 30 days after first use.

Keep away from children below 3 years. Small parts present a choking hazard – keep out of reach of children.

**⚠ If your ear canals are blocked, we recommend visiting an ENT specialist. After the ears have been professionally cleaned by a specialist we recommend flushing them regularly with the earClin earshower to prevent further earwax build-up.**

## 6. WHY IS EARCLIN EARSHOWER SAFE TO USE?

earClin earshower thoroughly flushes out the cerumen without any mechanical intervention. The rounded shape of the ear piece makes it feel pleasant against the user's ear. The dimensions prevent it from entering too far into the ear canal, making injuries virtually impossible, especially to the ear drum. At the same time, the ear piece acts as a spray head. Its patented design ensures that the flushing solution is optimally dispersed within the ear canal. The olive-shaped ear piece also features a mechanism to protect against excessive pressure.

Keep your earClin earshower within easy reach when taking a shower or bath.

## 7. WHY IS THE USE OF COTTON SWABS NOT ADVISED?

The use of cotton swabs is criticized by the medical profession as they may lead to cerumen impaction. The cotton swab is used blind and pushes the cerumen towards the ear drum. The cotton swab can also be responsible for perforating the ear drum.

## 8. CONTENT

- 1 earClin earshower
- Instruction for use

INGREDIENTS: SODIUM TALLOWATE, GLYCERIN, AQUA (WATER), SORBITOL, SUCROSE, PEG-4001, TETRASODIUM.

STORE AT ROOM TEMPERATURE, (10-25°C). USE WITHIN 8 MONTHS AFTER FIRST USE. KEEP EARCLIN EARSHOWER OUT OF THE SIGHT AND REACH OF CHILDREN.

## 9. CONTACT DETAILS

**Distributor**  
TheOTClab  
Paulus Potterstraat 20  
1071 DA Amsterdam  
The Netherlands  
www.theotclab.com

**Manufacturer**  
TheOTClab  
Paulus Potterstraat 20  
1071 DA Amsterdam  
The Netherlands  
www.theotclab.com

Last revision date: July 2018



10ml



earadult003